

heka*dental*



Heka Dental ApS
Baldershøj 38
DK-2635 Ishøj, Denmark
Phone +45 43 32 09 90
Fax +45 43 32 09 80
mail@heka-dental.dk
www.heka-dental.dk

Future

For further information - please contact your depot, service technicians or send back the enclosed card, then you will be contacted.

With reservation of technical changes and colour changes.

Kommunikator 21765 4/04 UK



DANISH
DESIGN
- MADE IN
DENMARK

FLEXIBILITY AND
INDIVIDUAL DESIGN
TURN YOUR DENTAL
UNIT INTO AN ACTIVE
PARTNER



2

QUALITY IS NOT A
CONSIDERATION
QUALITY IS THE
SOLUTION




UnicLine
- FEEL THE DIFFERENCE

3

COMPLETE CONTROL - WITH THE FOOT

The unique patented round foot control can be activated from all directions with either right or left foot releasing you from any fixed working posture. The foot control regulates the speed and the intensity of the chosen instruments. Via the middle ring of the foot control the spray functions can be selected, the strong, mechanical chip-blow activated or rotations of the micro motor reversed. The program has also 3 preselected speeds of the micromotor or oscillations of the ultrasonic. The top of the foot control can activate the operating light or the assistant call.


4

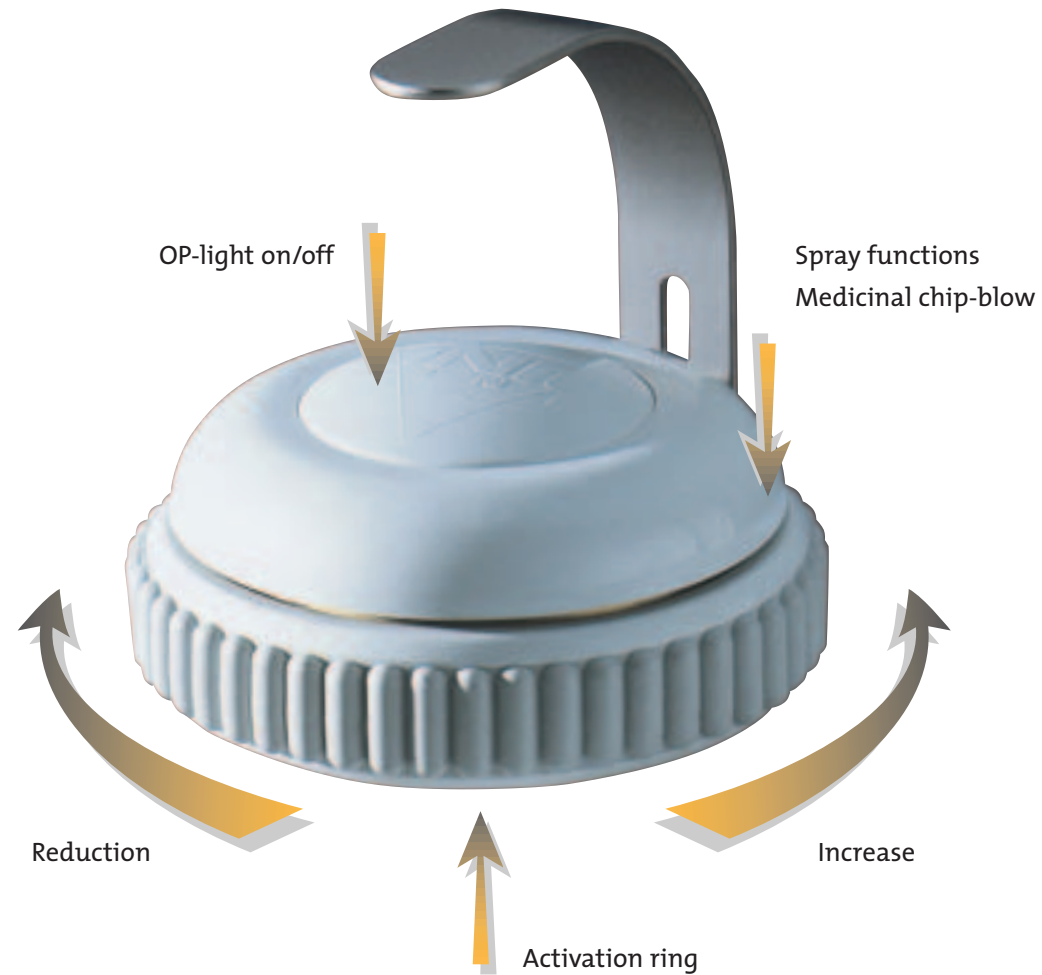

UnicLine can also be delivered with a conventional, variable foot control.



LEAVE THE POSSIBILITIES OPEN - EVERY DAY

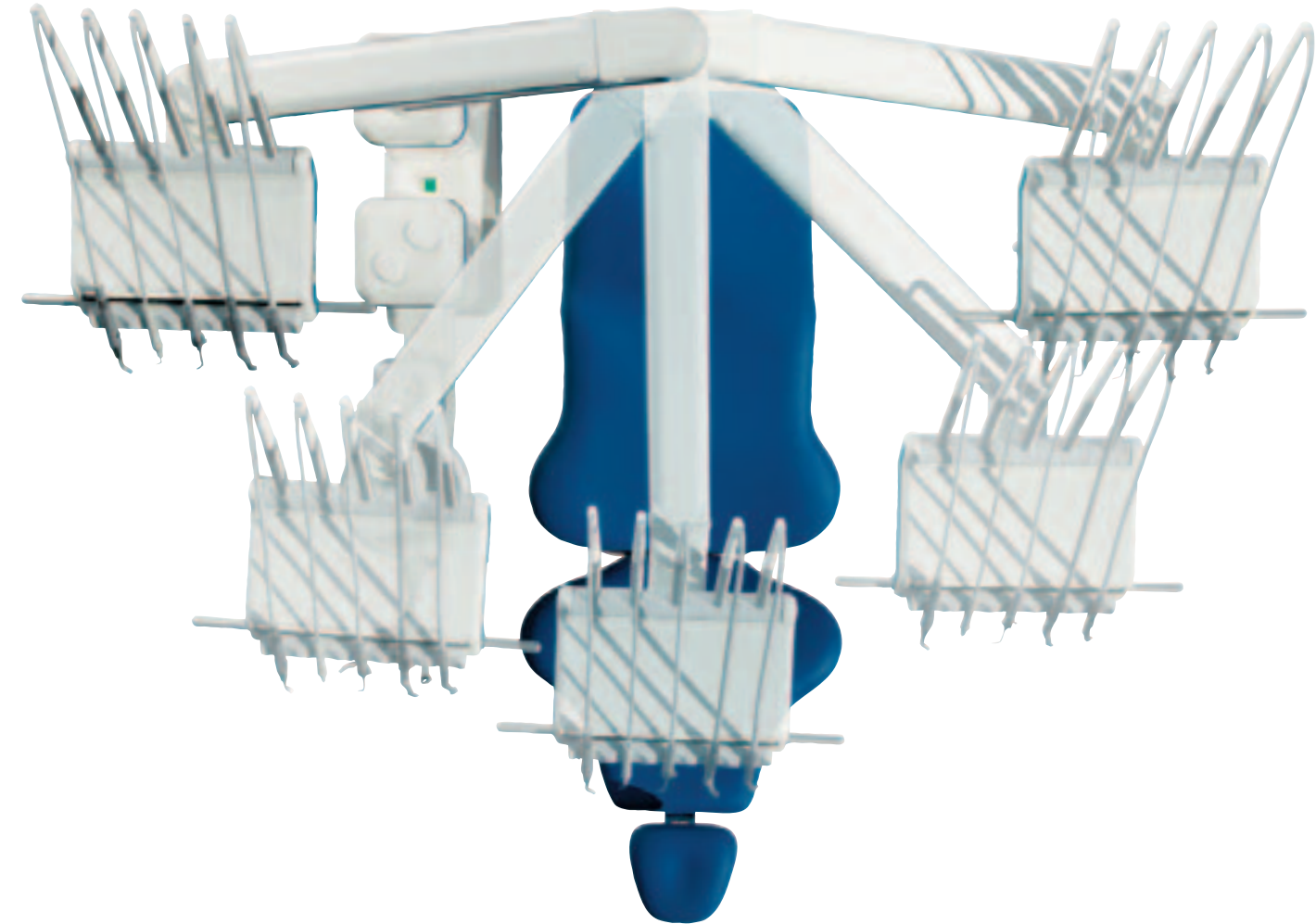

5



View

The multi-display gives you in a simple way a view over the current instrument programme.

Individual adjustment


The angle of the instrument holder can be individually adjusted for each instrument.

A wide range of options

The UnicLine 5D can come with from 3 to 6 balanced spring or sub-supported instruments individually placed on the bridge and a choice of more than 20 different instruments is available. All functions of instruments can be controlled from the foot control.

Finger Touch

The keyboard with the help functions is placed on the side of the instrument table and controls the timer function, the light curing time, the assistant call and the spittoon functions. Furthermore, the micro motor can be tested.

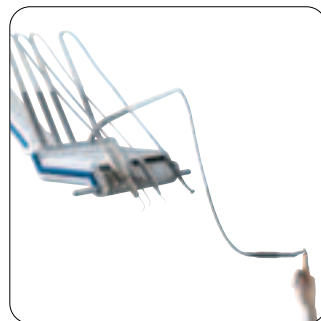


Ergonomics


On the balanced spring model each instrument is perfectly balanced and the extra long silicon hoses ensure ergonomically correct working postures.

WE ALL HAVE DIFFERENT IDEAS AND WISHES - FLEXIBILITY MEANS FOR INSTANCE ADAPTATION, THEREFORE EACH DENTAL UNIT IS ADAPTED TO THE INDIVIDUAL DENTIST'S NEEDS




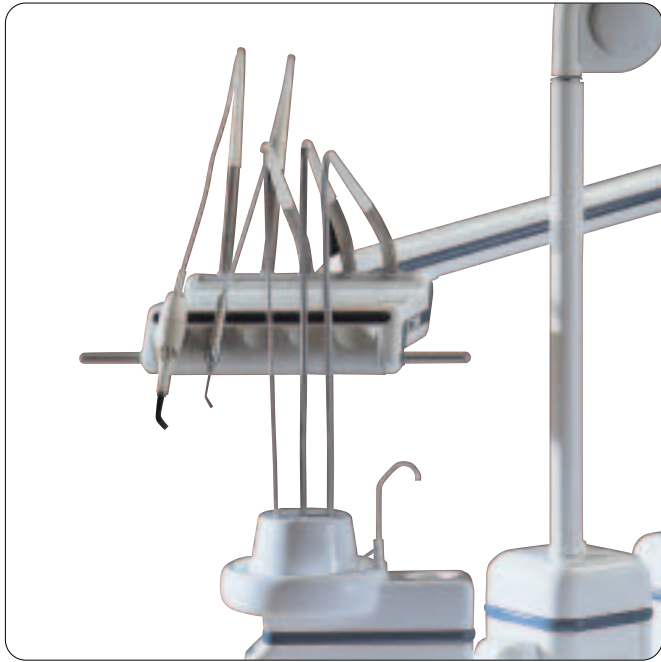

The easy removable aluminium handles can be autoclaved.

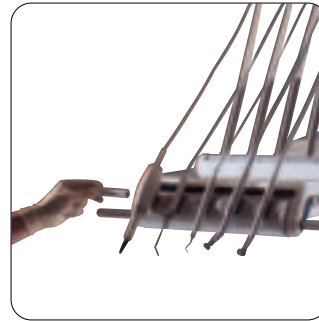

The instrument support can easily be removed for cleaning.

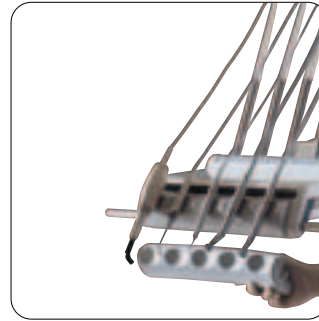


Rinsing of instruments

To ensure the supply of fresh water the UnicLine unit can be delivered with an automatic water rinsing system for the instruments.

Emptying of water canals

UnicLine can as option be delivered with a hygienic system, where the water is emptied from the tubings. We recommend to use this option, if the unit is not used for more than 2 days.

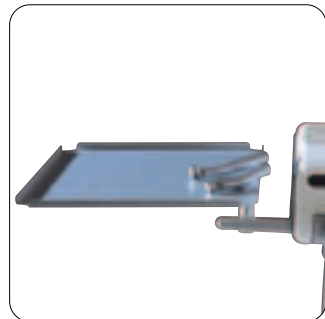


Unicline 5D
 - with single tray.
 - with double tray.
 - with X-ray monitor.



WHY COMPROMISE WITH THE MATERIALS, WHEN YOU CAN CHOOSE A DENTAL UNIT OF ALUMINIUM CASTING - THE SECURITY FOR A LIFE LONG QUALITY



UnicLine multimedia
UnicLine 5D with monitor
for patient friendly
showing of intraoral
photos, videos, digital
x-ray or other IT-media.

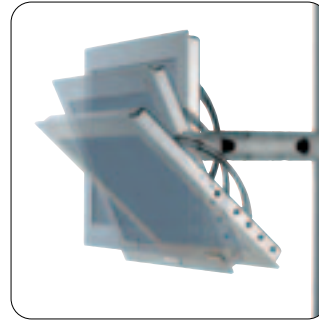

**FLEXIBILITY
MEANS
FOR INSTANCE
ADAPTATION**



10



11



UNICLINE 5D - AN
ELEGANT, FUNCTIONAL
AND TIMELESS DESIGN
WITH THOUGHT-
FULNESS FOR THE
ENVIRONMENT





UNICLINE 5D
CAN BE
ADAPTED TO
RIGHT- OR LEFT
HANDED DENTAL
TREATMENT.





LIGHT WITH BUILT-IN STABILITY




The UnicLine operating light gives a perfect treatment light with the lamp head placed on a self-balanced spring arm to secure a stable position.

FREE PLACEMENT

The Cart version offers the same functions as the UnicLine 5D unit. The unit is built on a solid aluminium profile with hidden wheels. The working height of the instrument table is manually adjustable for both sitting and standing treatment.



UnicLine Cart

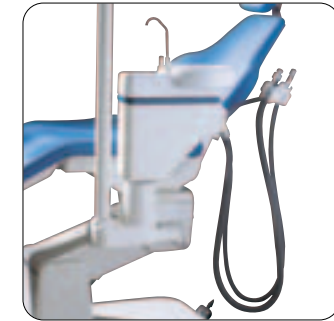

The chairside spittoon
The chair adapter is suited for the UnicLine patient chair and can be mounted with a turnable spittoon with suction, an extra tray table and an operating light as well as with a flexible telescopic arm.



UnicLine model: pillar


PATIENT HYGIENE

UnicLine 5D is available with the unique automatic suction hose cleaning system, which is activated by a single pressure.

Hereby the suction hoses are disinfected and a new treatment session can start. During the patient treatment the suction hoses can also be disinfected continuously.




UnicLine 5D is delivered with either adjustable suction valves or with hygienic aluminium suction valves.

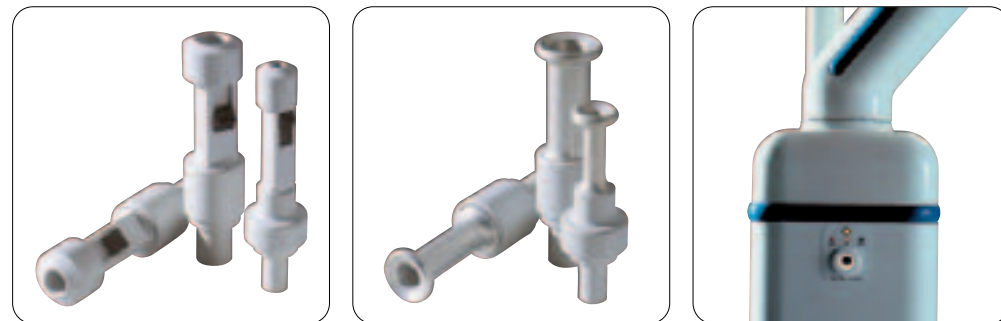

FLEXIBILITY

UnicLine 5D is supplied with a flexible telescopic arm which can be adjusted to the optimum working position, both if you work with or without an assistant. Can also be delivered with a multi-function-syringe or LED polymerisation lamp.

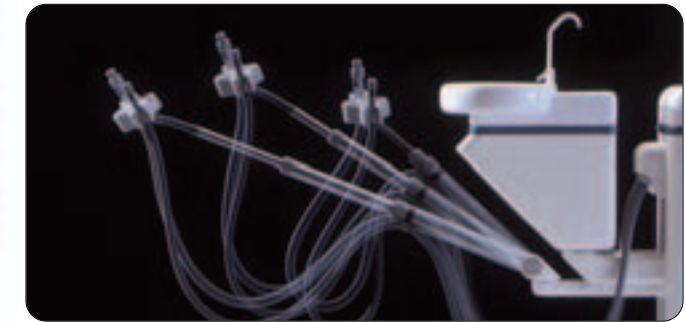

COMFORT

The spittoon can be turned and offers easy access for both 1 or 2 assistants. The spittoon upper part can easily be taken off.





Multi armrest

The UnicLine patient chair can be delivered with one or two armrests.



Joystick

Individual placement of up to 3 joy-sticks offers you and your assistant to maintain ideal working postures.



Easy to clean

The UnicLine upholstery is easy to dismount for cleaning.



A WIDE RANGE OF OPTIONS

The UnicLine patient chair is delivered with Soft upholstery in 9 different standard colours and as an option with SuperSoft upholstery in 4 different colours.

PATIENT COMFORT

The compensating movements of the back rest and the seat as well as the integrated elbow support ensure that the patient sits comfortably during the whole treatment.




Stable position

The double articulating neck rest improves the patient positioning, suitable for both adults and children.



UNICLINE MOBILE - COMPLETE UNIT

Full equipped mobile and
stationary treatment.

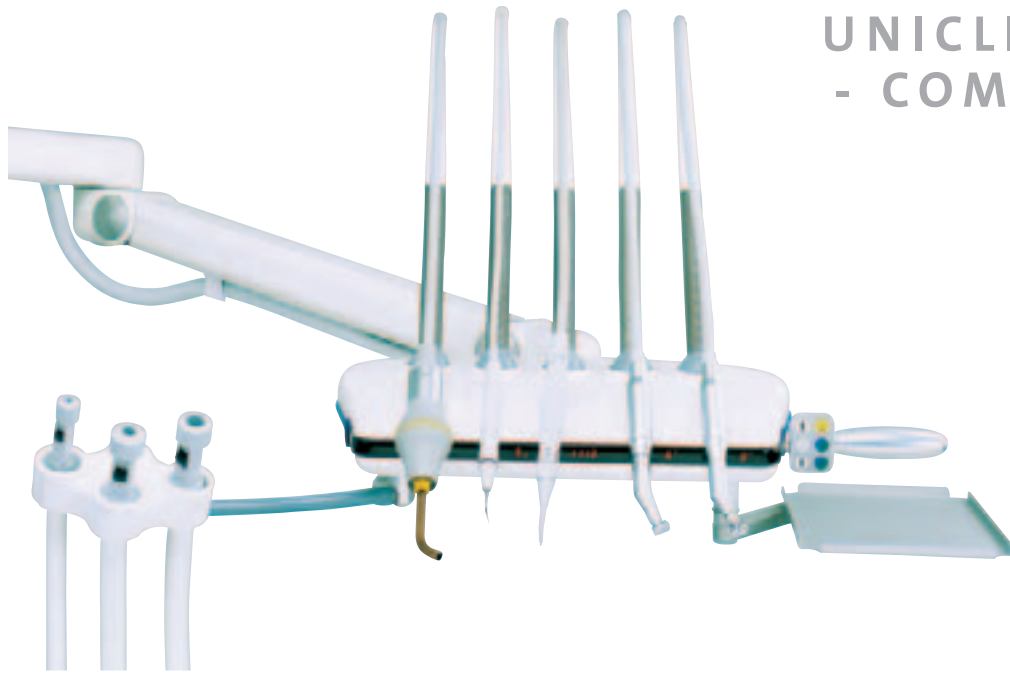

Instrument table with 2 motors,
3-function syringe, scaler and
polymerisation lamp.
The unit cabinet contains amalgam
separator, compressor and
separate tanks for fresh outlet water.
Only needs 230 VAC.



UNIT, INSTRUMENT TABLE, OP-LIGHT, OPERATION CHAIR AND X-RAYS

