

J1530 & J1531 Excedo Ultra-Bac - Kill Rates

Table 1: Test Summary (See Table 2 for organism details)

Organism Group	Dilution	
	1:10	1:40
Common Food poisoning Organisms TEST: EN 1276	30 seconds	5 minutes
Common Healthcare Infections TEST: EN 1276	30 seconds	5 minutes
Common Viruses I.e. HIV, Hepatitis B & C		5 minutes

Table 2: Laboratory Test Results

Microorganism	Class	Environment	Test Method	Contact time	Dilution	Result
<i>Escherichia coli</i> 0157 CCFRA 9913	Bacteria	Food	EN 1276	30 seconds	1:10	Pass
MRSA NCTC 12793	Bacteria	Healthcare	EN 1276	30 Seconds	1:10	Pass
<i>Staphylococcus aureus</i> NCIMB 9518	Bacteria	Food & Healthcare	EN 1276	30 seconds	1:10	Pass
<i>Salmonella typhimurium</i> NCTC 74	Bacteria	Food	EN 1276	30 seconds	1:10	Pass
<i>Enterococcus hirae</i> NCIMB 8192	Bacteria	Food	EN 1276	30 seconds	1:10	Pass
<i>Pseudomonas aeruginosa</i> NCIMB 10421	Bacteria	Healthcare	EN 1276	30 seconds	1:10	Pass
<i>Listeria monocytogenes</i> NCTC 11994	Bacteria	Food	EN 1276	30 seconds	1:10	Pass

<i>Escherichia coli</i> 0157 CCFRA 9913	Bacteria	Food	EN 1276	5 min	1:40	Pass
<i>MRSA</i> <i>NCTC 12793</i>	Bacteria	Healthcare	EN 1276	5 min	1:40	Pass
<i>Staphylococcus aureus</i> <i>NCIMB 9518</i>	Bacteria	Food & Healthcare	EN 1276	5 min	1:40	Pass
<i>Salmonella typhimurium</i> <i>NCTC 74</i>	Bacteria	Food	EN 1276	5 min	1:40	Pass
<i>Enterococcus hirae</i> NCIMB 8192	Bacteria	Food	EN 1276	5 min	1:40	Pass
<i>Pseudomonas aeruginosa</i> <i>NCIMB 10421</i>	Bacteria	Healthcare	EN 1276	5 min	1:40	Pass
<i>Listeria monocytogenes</i> <i>NCTC 11994</i>	Bacteria	Food	EN 1276	5 min	1:40	Pass
<i>H1N1 Swine Flu</i>	Virus	Healthcare	EN 14675	5 minutes	1:10	Pass
<i>Clostridium difficile</i> spores <i>NCTC 11209</i>	Bacteria	Healthcare	EN 13704	60 minutes	1:10	Pass
<i>Hepatitis B</i>	Virus	Healthcare	EN 14476	5 minutes	1:40	Pass
<i>Hepatitis C</i>	Virus	Healthcare	EN 14476	5 minutes	1:40	Pass
<i>HIV</i>	Virus	Healthcare	EN 14476	5 minutes	1:40	Pass
<i>Aspergillus niger</i> <i>AnFH 128/b</i>	Mould	Healthcare	EN 1650	15 minutes	1:10	Pass

<i>Aspergillus niger</i> <i>AnFH 128/b</i>	Mould	Healthcare	EN 1650	30 minutes	1:40	Pass
---	-------	------------	---------	------------	------	------

Issued JB January 2016