

Notes

- The products below are recommended for use on dental equipment.
- No dental equipment is completely resistant to chemical products. Therefore all products should be used within the manufacturers guidelines and with caution. Prolonged and excessive use of disinfecting chemicals may damage or decrease the life of dental equipment.
- Other contributing factors to the damage of dental equipment may include: lubricants, sterilants, ultraviolet light and excessive atmospheric conditions such as heat and humidity.
- The use of abrasive cleaners, scourers and paper products may cause unnecessary damage.
- Older dental equipment manufactured before the introduction of more resilient materials may be more susceptible to damage following the frequent and excessive application of chemical products.
- The use of waterproof barriers (covers) on all touch surfaces is recommended; these should be replaced in between patients.

Care and maintenance for dental equipment

PRODUCTS: *Please see individual manufacturers guidelines for dilution & usage									
	DURR FD 366	DURR FD 300 TOP WIPES	DURR FD 312	DURR FD 360	ALPRON	DURR MD 550	DURR MD 555	OROTOL PLUS	PULLJET
	Suitable for use in between patients	Suitable for use in between patients	Suitable for use at the end of every treatment day	Use daily if required	Water Line Additive – to be bottled water ONLY	As required	Suitable for Once or Twice a week	Suitable for use at the end of every treatment day – DURR Pump system ONLY	Suitable for use at the end of every treatment day – CATTANI Pump System ONLY
DELIVERY UNITS	✓	✓	✓	✗					
PAINT WORK	✓	✓	✓						
UPHOLSTERY	✗		✓	✓					
WATER LINES					✓				
SPITTOON BOWLS						✓			
DURR SUCTION HOSES							✓	✓	✗
CATTANI SUCTION HOSES								✗	✓

✓ Suitable product

✗ Caution: Could cause damage

*Always follow the label instructions on disinfectant products, taking note of 'reaction' times

*Always make sure the excess fluid is wiped clean with a non abrasive cloth

All recommended products are available from:

Eclipse Dental Engineering, Westview Court, 10 -14 Kent Road, Dartford, Kent DA1 2AJ

Guaranteed solutions...