


Tornado 1, Tornado 2, Tornado 4

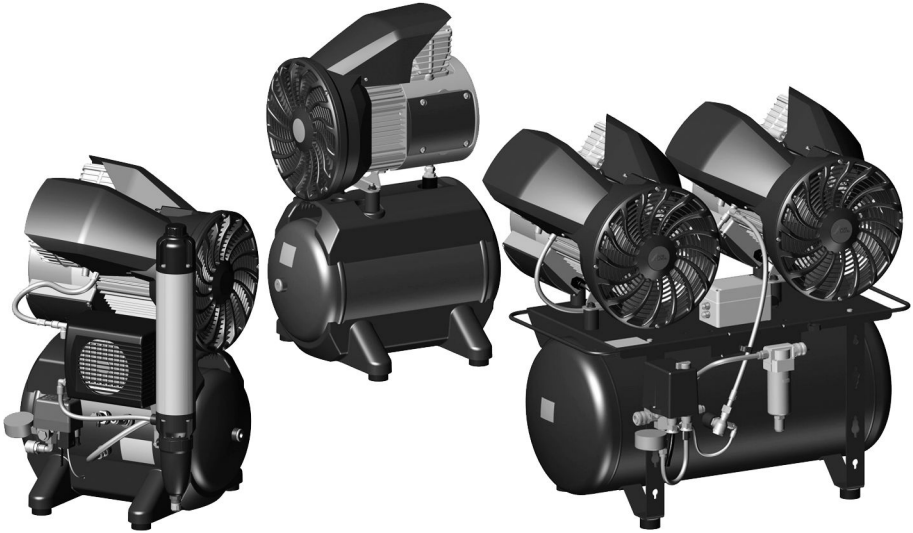

EN Installation and operating instructions

The current version of the installation and operating instructions is available in the Download Center:



<http://qr.duerdental.com/5180300004>

© 2025 DÜRR DENTAL SE

Contents



Important information

1	About this document	3
1.1	Warnings and symbols	3
1.2	Copyright information	4
2	Safety	4
2.1	Intended purpose	4
2.2	Intended use	4
2.3	Improper use	4
2.4	General safety information	5
2.5	Specialist personnel	5
2.6	Electrical safety	5
2.7	Notification requirement of serious incidents	5
2.8	Only use original parts	5
2.9	Transport	5
2.10	Disposal	6



Product description

3	Overview	7
3.1	Scope of delivery	7
3.2	Optional items	7
3.3	Wear parts and replacement parts	7
4	Technical data	8
4.1	Tornado 1	8
4.2	Tornado 1 with membrane drying unit	9
4.3	Tornado 1 with membrane drying unit	10
4.4	Tornado 2	11
4.5	Tornado 2 with membrane drying unit	12
4.6	Tornado 2 with membrane drying unit	13
4.7	Tornado 4	14
4.8	Tornado 4 with membrane drying unit	15

4.9	Classification of the compressed air supply system according to ISO 22052	17
4.10	Filter	17
4.11	Ambient conditions	17
4.12	Distance between rubber feet	18
4.13	Type plate	18
4.14	Evaluation of conformity	19
5	Function	20
5.1	Tornado 1	20
5.2	Tornado 2 with membrane drying unit	20
5.3	Tornado 4 Type 42803	21
5.4	Tornado 4 Type 42823	22



Assembly

6	Requirements	23
6.1	Installation/setup room	23
6.2	Setup	23
6.3	Information about electrical connections	23
7	Transport	24
8	Installation	24
8.1	Remove the transport locks	24
8.2	Establishing the compressed air connection	24
8.3	Place a collector tray underneath	25
8.4	Electrical connections	25
9	Commissioning	27
9.1	Checking the switch-on/cut-off pressure	27
9.2	Draining the condensation water	27
10	Adjustment options	28
10.1	Adjusting the pressure switch	28
11	Circuit diagrams	30
11.1	Tornado 1 / Tornado 2	30
11.2	Tornado 4	32



Usage

12 Operation	33
12.1 Switching the unit on/off	33
13 Maintenance	34
13.1 Maintenance schedule	34
13.2 Wear parts and replacement parts	35
13.3 Replacing the air intake filter	36
13.4 Replace the fleece filter in the condensate separator	37
13.5 Replacing the filter of the membrane drying unit	38
13.6 Checking the safety valve	38
14 Taking out of use	39
14.1 Taking the unit out of use	39
14.2 Storage of the unit	40



Troubleshooting

15 Tips for operators and service technicians	41
--	----



Appendix

16 Handover record	43
17 Country representatives	44

Important information

1 About this document

These installation and operating instructions represent part of the unit.



The manufacturer and the distributor will not offer any guarantee or accept any liability for the safe operation and the safe functioning of the unit if the instructions and information in these installation and operating instructions are not complied with.

The German version of the installation and operating instructions is the original manual. All other languages are translations of the original manual. These installation and operating instructions apply to:

Tornado 1

REF: 5180300001, 5180300003, 5182300001, 5182300003, 5186300001

Tornado 2

REF: 5280300001, 5280300003, 5282300001, 5282300003, 5286300001

Tornado 4

REF: 4280300001, 4282300001, 4282300003

1.1 Warnings and symbols

Warnings

The warnings in this document are intended to draw your attention to possible injury to persons or damage to machinery.

The following warning symbols are used:



General warning symbol



Warning – dangerous high voltage



Warning – hot surfaces



Warning - automatic start-up of the unit

The warnings are structured as follows:



SIGNAL WORD

Description of the type and source of danger

Here you will find the possible consequences of ignoring the warning

- Follow these measures to avoid the danger.

The signal word differentiates between four levels of danger:

- **DANGER**
Immediate danger of severe injury or death
- **WARNING**
Possible danger of severe injury or death
- **CAUTION**
Risk of minor injuries
- **NOTICE**
Risk of extensive material/property damage

Other symbols

These symbols are used in the document and on or in the unit:



Note, e.g. specific instructions regarding efficient and cost-effective use of the unit.



Refer to the accompanying electronic documents.



Disconnect all power from the unit.



Air



Filter symbol



Dispose of correctly in accordance with EU Directive 2012/19/EU (WEEE).



CE labelling with the number of the notified body



Ukrainian conformity mark with registration number



Conformity mark for the United Kingdom of Great Britain and Northern Ireland, with the number of the designated authority



Authorised representative for Switzerland

REF Order number

SN Serial number

MD Medical device

HIBC Health Industry Bar Code (HIBC)

 Manufacturer

1.2 Copyright information

All circuits, processes, names, software programs and units mentioned in this document are protected by copyright.

The Installation and Operating Instructions must not be copied or reprinted, neither in full nor in part, without written authorisation from the copyright owner.

2 Safety

The unit has been developed and designed in such a way that dangers are effectively ruled out if used in accordance with the Intended Use.

Despite this, the following residual risks can remain:

- Personal injury due to incorrect use/misuse
- Personal injury due to mechanical effects
- Personal injury due to electrical shock
- Personal injury due to radiation
- Personal injury due to fire
- Personal injury due to thermal effects on skin
- Personal injury due to lack of hygiene, e.g. infection



WARNING

The development of emphysema

Soft tissue can be damaged as a result of careless handling.

- › Do not dwell in the area being treated for any longer than is necessary.

2.1 Intended purpose

The compressor is designed to supply compressed air for dental applications.

2.2 Intended use

The air supplied by the compressor is suitable for driving dental tools.

The compressed air generated by the compressor is delivered to the pipeline system of the surgery. The entire compressed air system must be designed in such a way that the quality of the compressed air generated by the compressor is not impaired.

With this prerequisite, the air provided by the compressor is also suitable for blow-drying tooth preparations.

2.3 Improper use

Any use of this appliance / these appliances above and beyond that described in the Installation and Operating Instructions is deemed to be incorrect usage. The manufacturer cannot be held liable for any damage resulting from incorrect usage. The operator will be held liable and bears all risks.

**WARNING****Risk of explosion due to ignition of combustible materials**

- Do not operate the unit in any rooms in which inflammable mixtures may be present, e.g. in operating theatres.

- The unit is not suitable for providing an air supply to respirators.
- This unit is not suitable for drawing up fluids or for compressing aggressive gases or potentially explosive gases.

2.4 General safety information

- Always comply with the specifications of all guidelines, laws, and other rules and regulations applicable at the site of operation for the operation of this unit.
- Check the function and condition of the unit prior to every use.
- Do not convert or modify the unit.
- Comply with the specifications of the Installation and Operating Instructions.
- The Installation and Operating Instructions must be accessible to all operators of the unit at all times.

2.5 Specialist personnel**Operation**

Unit operating personnel must ensure safe and correct handling based on their training and knowledge.

- Instruct or have every operator instructed in handling the unit.

The following groups are not permitted to operate or use a commercially operated unit:

- People without the necessary experience and knowledge
- People with reduced physical, sensory or mental capabilities
- Children

Installation and repairs

- The manufacturer recommends that installation, readjustments, alterations, upgrades and repairs be carried out either by the manufacturer itself or by a qualified specialist authorised by the manufacturer.

2.6 Electrical safety

- Observe and comply with all the relevant electrical safety regulations when working on the unit.
- Replace any damaged cables or plugs immediately.

2.7 Notification requirement of serious incidents

The operator/patient is required to report any serious incident that occurs in connection with the device to the manufacturer and to the competent authority of the Member State in which the operator and/or patient is established/resident.



Notify the manufacturer of any serious incidents under the following email address: incidents@duerrdental.com.

2.8 Only use original parts

- Only use accessories and optional articles named or authorised by the manufacturer.
- Only use only original wear parts and replacement parts.



The manufacturer and distributor accept no liability for damages or injury resulting from the use of non-approved accessories, optional accessories, or from the use of non-original wear parts or replacement parts.

The use of non-approved accessories, optional accessories or non-genuine wear parts / replacement parts (e.g. mains cables) can have a negative effect in terms of electrical safety and EMC.

2.9 Transport

The original packaging provides optimum protection for the unit during transportation.

If required, the original packaging for the unit can be ordered.



The manufacturer and the distributor do not accept liability, even during the warranty period, for damage during transportation due to improper packaging.

- Only transport the unit in its original packaging.
- Keep the packing materials out of the reach of children.

2.10 Disposal



The unit must be disposed of properly. Within the European Union, the unit must be disposed of in accordance with EU Directive 2012/19/EU (WEEE).

If you have any questions about the correct disposal of parts, please contact your dental trade supplier.



An overview of the waste keys for Dürr Dental products can be found in the download area:



<http://qr.duerdental.com/P007100155>



Product description

3 Overview

3.1 Scope of delivery

The following items are included in the scope of delivery (possible variant-specific deviations due to country-specific requirements and/or import regulations):

<i>Tornado 1</i>	<i>5180300001</i>
<i>Tornado 1</i>	<i>5180300003</i>
<i>Tornado 2</i>	<i>5280300001</i>
<i>Tornado 2</i>	<i>5280300003</i>
<i>Tornado 4</i>	<i>4280300001</i>

- Compressor
- Fabric reinforced hose
- Hose nozzle
- Hose clamp
- Vibration dampers
- Cable ties
- Mains cable
- Short information

<i>Tornado 1 with membrane drying unit</i>	<i>5182300001</i>
--	-------------------

<i>Tornado 1 with membrane drying unit</i>	<i>5182300003</i>
--	-------------------

<i>Tornado 1 with membrane-drying unit, sound-insulated</i>	<i>5186300001</i>
---	-------------------

<i>Tornado 2 with membrane drying unit</i>	<i>5282300001</i>
--	-------------------

<i>Tornado 2 with membrane drying unit</i>	<i>5282300003</i>
--	-------------------

<i>Tornado 2 with membrane-drying unit, sound-insulated</i>	<i>5286300001</i>
---	-------------------

<i>Tornado 4 with membrane drying unit</i>	<i>4282300001</i>
--	-------------------

<i>Tornado 4 with membrane drying unit</i>	<i>4282300003</i>
--	-------------------

- Compressor
- Fabric reinforced hose
- Hose nozzle
- Hose clamp
- Vibration dampers
- Cable ties
- Mains cable
- Short information
- Collector tray

3.2 Optional items

The following optional items can be used with the device:

Pressure reducer	6040-992-00
Virus bacteria filter	1650100172
Wooden cabinet for sound insulation	5150-500-00

3.3 Wear parts and replacement parts

The following working parts need to be changed at regular intervals (refer to the "Maintenance" section):

Air intake filter	5180-982-00
Fine filter	1610-121-00
Virus bacteria filter	1650100172
Coalescence filter	1650200323
Fleece filter	4280-982-00
Vibration damper set Tornado 1	5180100165
Vibration damper set Tornado 2	5280100033
Vibration damper set Tornado 4	4280100045
Cup seal repair set	5180100157



To configure the required filters or filter sets, you can also use our filter configurator at:

www.duerrdental.com/filterkonfigurator



Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.



Information about replacement parts can be found on the website portal for specialist dealers under:

www.duerrdental.net



If the mains cable of this unit is damaged it must only be replaced by an original mains cable from the manufacturer.



If the mains cable of this unit is damaged, it must be replaced by an adequate mains cable (H05VV-F, or abbreviation 60227 IEC 53, minimum cable cross-section 1 mm²).

4 Technical data

4.1 Tornado 1

Electrical data		5180300001 5180300003	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	3.7	4.0
Rated power	kW	0.8	0.9
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	10	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	130	155
Delivery at 5 bar (0.5 MPa)	l/min	68	78
Delivery at 7 bar (0.7 MPa)	l/min	55	59
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	104	90
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	69 x 48 x 41	
Weight	kg	32	
Noise level **	dB(A)	64	68
With wooden cabinet	dB(A)	47	49

* Values without accessories and add-on parts

** In accordance with ISO 3744

Classification	
Medical Device Class (MDR)	Ila

4.2 Tornado 1 with membrane drying unit

Electrical data		5182300001 5182300003	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	3.9	4.2
Rated power	kW	0.8	1.0
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	10	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	130	155
Delivery at 5 bar (0.5 MPa)	l/min	62	70
Delivery at 7 bar (0.7 MPa)	l/min	53	61
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	130	113
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	69 x 49 x 47	
Weight	kg	38	
Noise level **	dB(A)	64	68
With wooden cabinet	dB(A)	47	49

* Values without accessories and add-on parts

** In accordance with ISO 3744

Requirements of ISO 22052	
Quality of dental air: *	
Particle class 2:	Compliant
Humidity class 4:	Compliant
Residual oil class 2:	Compliant
* measured with virus-bacteria filter	

Classification	
Medical Device Class (MDR)	Ila

4.3 Tornado 1 with membrane drying unit

Electrical data		5186300001	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	3.9	4.3
Rated power	kW	0.8	1.0
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	10	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	130	155
Delivery at 5 bar (0.5 MPa)	l/min	62	70
Delivery at 7 bar (0.7 MPa)	l/min	53	61
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	130	113
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	84 x 63 x 60	
Weight	kg	49	
Noise level **	dB(A)	58	60

* Values without accessories and add-on parts

** In accordance with ISO 3744

Requirements of ISO 22052	
Quality of dental air: *	
Particle class 2:	Compliant
Humidity class 4:	Compliant
Residual oil class 2:	Compliant

* measured with virus-bacteria filter

Classification	
Medical Device Class (MDR)	IIa

4.4 Tornado 2

Electrical data	5280300001		
	5280300003		
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	7.5	7.1
Rated power	kW	1.4	1.6
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	16	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	260	315
Delivery at 5 bar (0.5 MPa)	l/min	134	153
Delivery at 7 bar (0.7 MPa)	l/min	117	136
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	57	49
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	67 x 48 x 41	
Weight	kg	38	
Noise level **	dB(A)	68	71
With wooden cabinet	dB(A)	51	53

* Values without accessories and add-on parts

** ISO 3744

Classification	
Medical Device Class (MDR)	IIa

4.5 Tornado 2 with membrane drying unit

Electrical data		5282300001 5282300003	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	7.5	7.1
Rated power	kW	1.5	1.6
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	16	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	260	315
Delivery at 5 bar (0.5 MPa)	l/min	118	137
Delivery at 7 bar (0.7 MPa)	l/min	102	119
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	74	64
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	65 x 49 x 48	
Weight	kg	44	
Noise level **	dB(A)	68	71
With wooden cabinet	dB(A)	51	53

* Values without accessories and add-on parts

** In accordance with ISO 3744

Requirements of ISO 22052	
Quality of dental air: *	
Particle class 2:	Compliant
Humidity class 4:	Compliant
Residual oil class 2:	Compliant

* measured with virus-bacteria filter

Classification	
Medical Device Class (MDR)	Ila

4.6 Tornado 2 with membrane drying unit

Electrical data		5286300001	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 MPa)	A	7.5	7.1
Rated power	kW	1.4	1.6
Motor protection	Motor winding overheat protector		
Type of protection	IP 21		
Mains fuses *	A	16	

* Circuit breaker fuse characteristics B, C or D in acc. with EN 60898-1

General technical data			
Pressure tank volume	l	20	
Suction power, approx.	l/min	260	315
Delivery at 5 bar (0.5 MPa)	l/min	113	132
Delivery at 7 bar (0.7 MPa)	l/min	96	113
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	76	66
Duty cycle	%	100	
Start-up pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	84 x 63 x 60	
Weight	kg	57	
Noise level **	dB(A)	60	62

* Values without accessories and add-on parts

** In accordance with ISO 3744

Requirements of ISO 22052	
Quality of dental air: *	
Particle class 2:	Compliant
Humidity class 4:	Compliant
Residual oil class 2:	Compliant

* measured with virus-bacteria filter

Classification	
Medical Device Class (MDR)	IIa

4.7 Tornado 4

Electrical data		4280300001	
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 Mpa)	A	14.7	14.4
Rated power	kW	2.9	3.3
Motor protection	Motor winding overheating protector and thermal protection switch		
Type of protection	IP 21		
Mains fuses *	A	16	

* LS switch fuse characteristics C and D in accordance with DIN EN60898

General data			
Pressure tank volume	l	50	
Suction power, approx.	l/min	525	630
Delivery at 5 bar (0.5 MPa)	l/min	260	296
Delivery at 7 bar (0.7 MPa)	l/min	229	261
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	75	65
Duty cycle	%	100 (S1)	
Cut-In pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	73 x 78 x 54	
Weight	kg	84	
Noise level **	dB(A)	73	75

* Values without accessories and add-on parts

** In accordance with ISO 3744

Classification	
Medical Device Class (MDR)	Ila

4.8 Tornado 4 with membrane drying unit

Electrical data		4282300001	4282300003
Rated voltage	V	230	
Mains frequency	Hz	50	60
Nominal current at 8 bar (0.8 Mpa)	A	14.7	14.4
Rated power	kW	2.9	3.3
Motor protection	Motor winding overheating protector and thermal protection switch		
Type of protection	IP 21		
Mains fuses *	A	16	
Max. permissible mains impedance in accordance with EN 61000-3-11	Ω	0.0956	

* LS switch fuse characteristics C and D in accordance with DIN EN60898

General data			
Pressure tank volume	l	50	
Suction power, approx.	l/min	525	630
Delivery at 5 bar (0.5 MPa)	l/min	214	251
Delivery at 7 bar (0.7 MPa)	l/min	188	219
Pressure build-up phase 0 - 7.5 bar (0 - 0.75 MPa) c.	s	100	86
Duty cycle	%	100 (S1)	
Cut-In pressure	bar (MPa)	6 (0.6)	
Cut-out pressure	bar (MPa)	7.8 (0.78)	
Cut-off pressure, max. adjustable	bar (MPa)	8 (0.8)	
Safety valve, maximum permissible operating pressure	bar (MPa)	10 (1)	
Dimensions (H x W x D) *	cm	73 x 78 x 61	
Weight	kg	90	
Noise level **	dB(A)	73	75

* Values without accessories and add-on parts

** In accordance with ISO 3744

Requirements of ISO 22052	
Quality of dental air: *	
Particle class 2:	Compliant
Humidity class 4:	Compliant
Residual oil class 2:	Compliant
* measured with virus-bacteria filter	

Classification

Medical Device Class (MDR)

IIa

4.9 Classification of the compressed air supply system according to ISO 22052

In accordance with ISO 22052, central compressed air supply systems are classified into different types depending on the type of lubrication process of the compressor. The compressor system described in these instructions has a compressor that is not lubricated with oil and is therefore classified as Type 2. The oil content of the ambient air is not changed by the central compressed air supply system.

4.10 Filter

Filter mesh size		
Air intake filter	µm	3
Fine filter	µm	3
Virus bacteria filter	µm	0.01
Coalescence filter	µm	0.01
Fleece filter	µm	5

4.11 Ambient conditions

Ambient conditions during storage and transport		
Temperature	°C	-10 - +55
Relative humidity	%	< 95
Ambient conditions during operation		
Temperature	°C	+10 - +40
Ideal temperature	°C	+10 - +25
Relative humidity	%	< 95
Height above sea level	m	< 2000



NOTICE


Risk of overheating due to insufficient ventilation

The units generates heat. Possibility of heat damage and/or reduced service life of the unit.

- › Do not cover the unit.
- › Install a fan for auxiliary ventilation in rooms where ambient temperatures exceed ≥ 40 °C while the unit is in operation.


4.12 Distance between rubber feet

Tornado 1 / Tornado 2

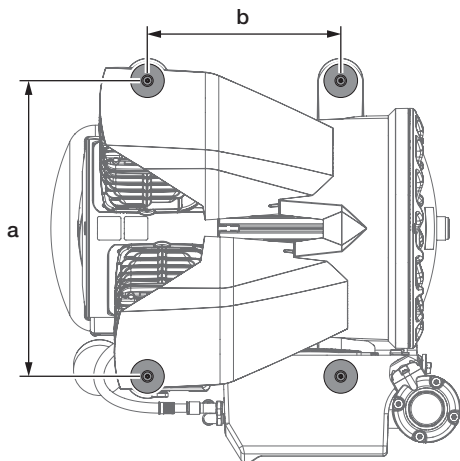


	a (cm)	b (cm)
20 l	35	23

With noise reducing hood

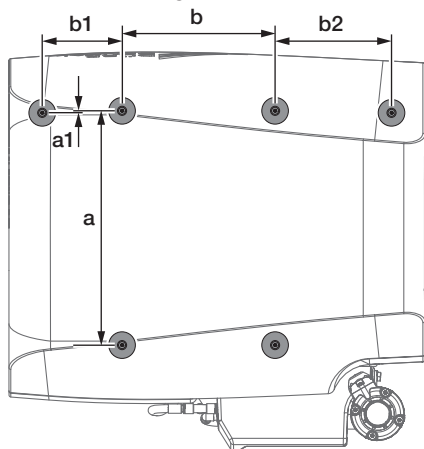

	A (cm)	a1 (cm)	b (cm)	b1 (cm)	b2 (cm)
20 l	35	0.5	23	17.5	12

Tornado 4

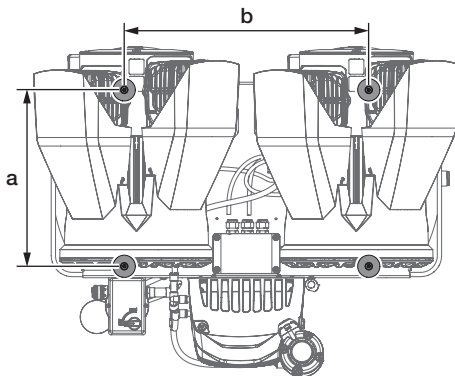


	a (cm)	b (cm)
50 l	32.5	45

4.13 Type plate

Tornado 1 / Tornado 2


The type plate for the complete system is located on the pressure tank.

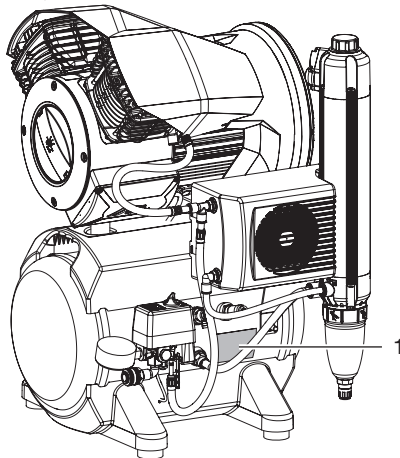

1 Type plate for the complete system

Tornado 4


The type plate for the complete system is located on the pressure tank.

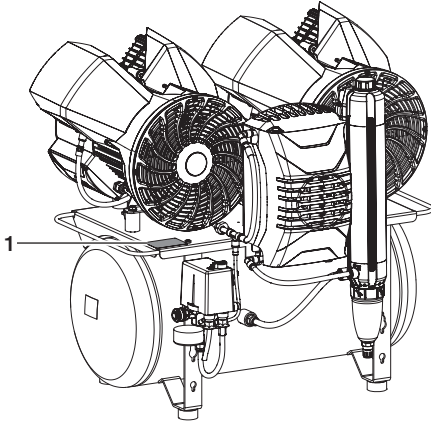

1 Type plate for the complete system

Compressor unit


The type plate of the compressor unit is located on the crankcase below the cylinder.

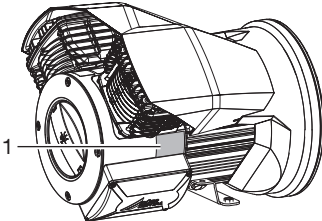

1 Compressor unit type plate

Membrane drying unit

The type plate of the membrane drying unit is located on the membrane drying unit.

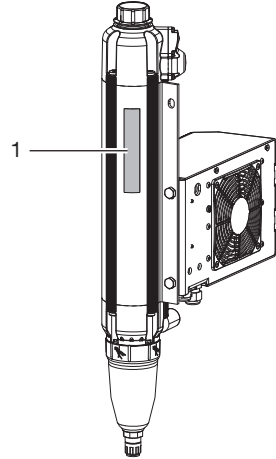


1 Membrane drying unit type plate

4.14 Evaluation of conformity

This device has been subjected to conformity acceptance testing in accordance with the current relevant European Union guidelines. This equipment conforms to all relevant requirements.

5 Function

5.1 Tornado 1

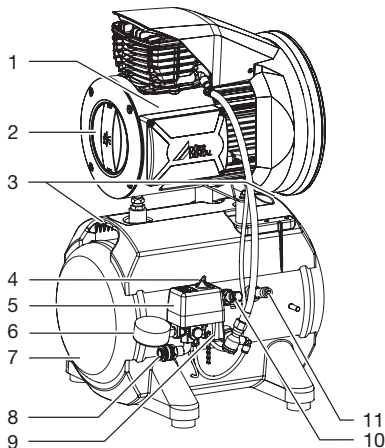

- 1 Compressor unit
- 2 Air intake filter
- 3 Carry handles
- 4 On/off switch
- 5 Pressure switch
- 6 Pressure gauge/display
- 7 Pressure vessel
- 8 Compressed air connection (quick release coupling)
- 9 Mains connection
- 10 Condensate drain valve
- 11 Safety valve



NOTICE


Risk of corrosion of the unit

Moisture can lead to premature corrosion.

➤ Retrofit a membrane drying unit.

The compressor unit draws in atmospheric air and compresses this air oil-free. The oil-free compressed air is then transported directly to the pressure tank. The oil-free and hygienic air is made available to the consumers in the pressure tank.


5.2 Tornado 2 with membrane drying unit

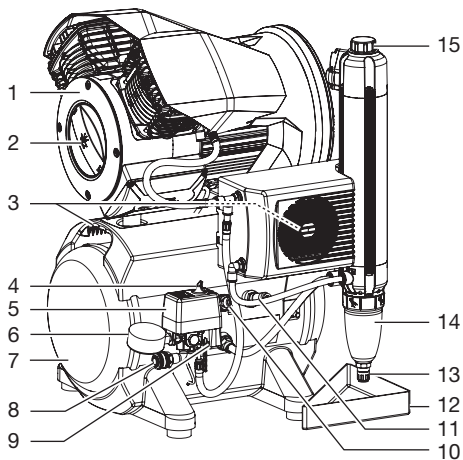

- 1 Compressor unit
- 2 Air intake filter
- 3 Carry handles
- 4 On/off switch
- 5 Pressure switch
- 6 Pressure gauge/display
- 7 Pressure vessel
- 8 Compressed air connection (quick release coupling)
- 9 Mains connection
- 10 Condensate drain valve
- 11 Safety valve
- 12 Collector tray
- 13 Automatic/manual condensate drain valve, membrane drying unit
- 14 Coalescing filter
- 15 Fine or virus bacteria filter

The compressor unit draws in atmospheric air and compresses this air oil-free. It then transports the oil-free compressed air to the membrane drying unit. The cooler and the membrane dryer extract moisture from the compressed air. The oil-free, hygienic and dry air is stored in the pressure tank ready for use in connected devices.

5.3 Tornado 4 Type 42803...

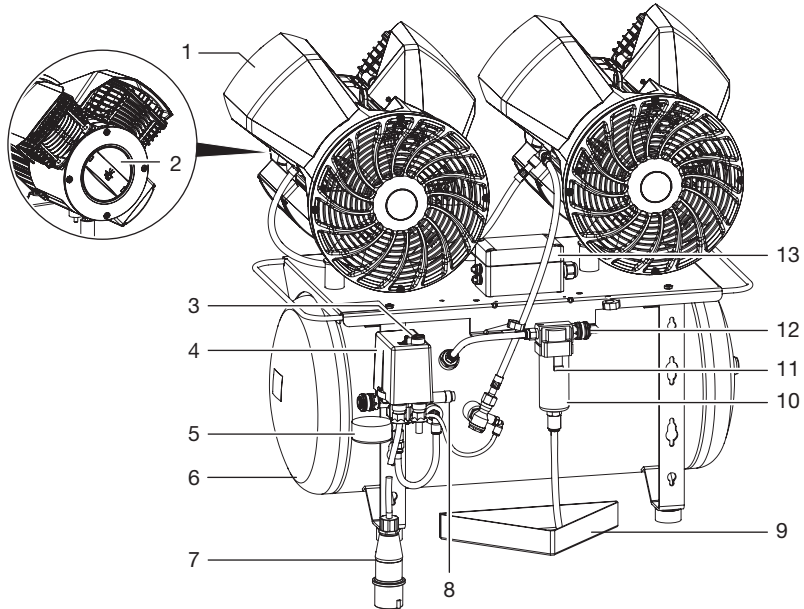

- 1 Compressor unit
- 2 Air intake filter
- 3 On/off switch
- 4 Pressure switch
- 5 Pressure gauge/display
- 6 Pressure vessel
- 7 Mains connection for CEE connector 230 V / 16 A
- 8 Safety valve
- 9 Collector tray
- 10 Condensate separator
- 11 Fleece filter in condensate separator
- 12 Compressed air connection (quick release coupling)
- 13 Thermal protection switch



NOTICE

Risk of corrosion of the unit


Moisture can lead to premature corrosion.

➤ Retrofit a membrane drying unit.

The compressor unit draws in atmospheric air and compresses this air oil-free. The oil-free compressed air is then transported directly to the pressure tank. The oil-free and hygienic air is made available to the consumers in the pressure tank.

An automatic condensate separator is installed between the pressure tank and the compressed air connection. When removing air, the condensate is collected from the pressure tank into the condensate separator. As soon as a specific fill level is reached in the condensate separator, the condensate is discharged automatically.

5.4 Tornado 4 Type 42823...

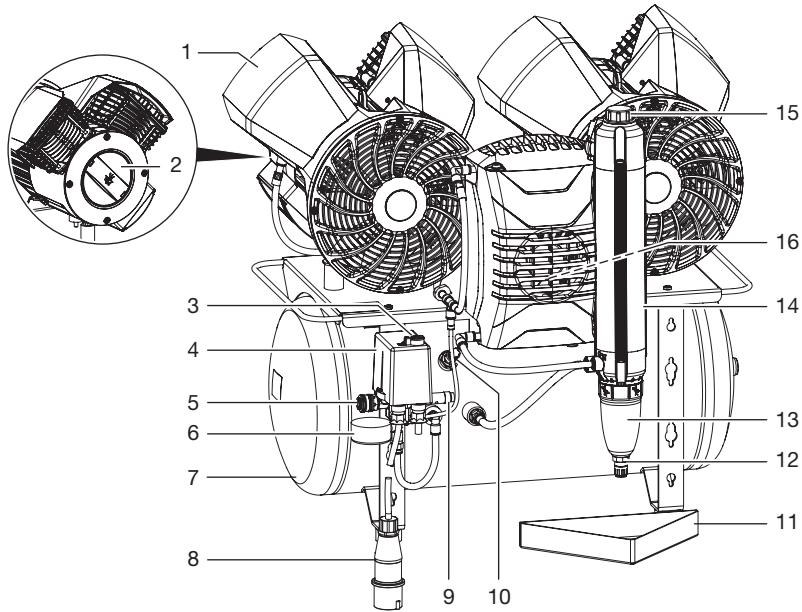


- 1 Compressor unit
- 2 Air intake filter
- 3 On/off switch
- 4 Pressure switch
- 5 Compressed air connection (quick release coupling)
- 6 Pressure gauge/display
- 7 Pressure vessel
- 8 Mains connection for CEE connector 230 V / 16 A
- 9 Safety valve
- 10 Condensate drain valve
- 11 Collector tray
- 12 Automatic/manual condensate drain valve, membrane drying unit
- 13 Coalescing filter
- 14 Membrane drying unit
- 15 Fine or virus bacteria filter
- 16 Thermal protection switch (behind the membrane drying unit)

The compressor unit draws in atmospheric air and compresses this air oil-free. It then transports the oil-free compressed air to the membrane drying unit. The cooler and the membrane dryer extract moisture from the compressed air. The oil-free, hygienic and dry air is stored in the pressure tank ready for use in connected devices.



Assembly

6 Requirements


 The unit must not be set up or operated within the vicinity of the patients (within a radius of 1.5 m).

The unit can be installed either at the same level as the surgery room, on a floor below (e.g. cellar) or under the roof.

Due to the amount of noise generated, we recommend that the unit is installed in an adjoining room.

The pipes provided on-site must at least meet the country-specific requirements for drinking water.

The compressed air network to which the unit is connected must be designed for the maximum pressure of the unit (10 bar).

 Further information can be found in the planning information leaflet for compressed air, available separately.

6.1 Installation/setup room

The room chosen for set up must fulfil the following requirements:

- Closed, dry, well-ventilated room
- Should not be a room made for another purpose (e. g. boiler room or wet cell)
- No installation room where the air is contaminated with oil from other systems.
- If the unit is installed in a machine room, e. g. utility room or basement, the requirements of ISO 22052 must be taken into consideration.




NOTICE

Risk of overheating due to insufficient ventilation


The units generates heat. Possibility of heat damage and/or reduced service life of the unit.

- › Do not cover the unit.
- › Install a fan for auxiliary ventilation in rooms where ambient temperatures exceed ≥ 40 °C while the unit is in operation.

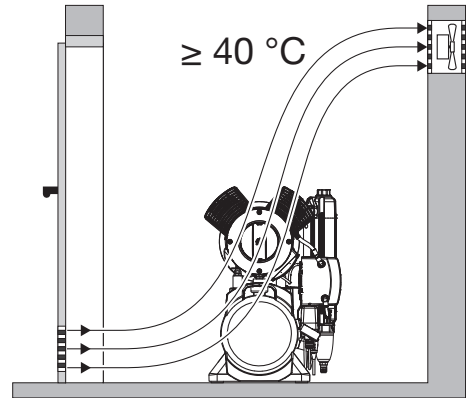

6.2 Setup

The following conditions must be taken into account for installation:

 The air is filtered when it is sucked in. This does not alter the composition of the air. For this reason it is important to keep the sucked-in air free of harmful substances (e.g. do not suck in exhaust gases or contaminated exhaust air).

- Clean, level and sufficiently stable subsurface (note the weight of the unit).
- Type plate easy to read.
- Unit easy to access for operation and maintenance.
- Easy-to-access power outlet to which the unit is connected.
- Maintain sufficient distance to the wall (at least 20 cm).
- The compressed air pipe should be routed as closely as possible to the place of installation (note the length of the hose supplied).

6.3 Information about electrical connections

- Ensure that the electrical connections to the mains power supply are established in accordance with current valid national and local regulations and standards governing the installation of low voltage units in medical facilities.
- Observe the current consumption of the devices that are to be connected.

7 Transport



WARNING

Risk of explosion of the pressure tank and pressure hoses

- › The pressure tank and the pressure hoses must be vented before they are stored or transported.

- Protect the unit against moisture, dirt and extreme temperatures during transport.
- Always make sure that the condensate collector chamber is empty before transporting the unit.
- Always transport the unit in an upright position.
- Only transport the unit using the transport handles provided.
- Check the unit for transport damage.

8 Installation

8.1 Remove the transport locks

For safe transport, the unit is secured with foam padding blocks and a transport strap.


1. Cut and remove the transport strap.
2. Remove the foam padding blocks.

8.2 Establishing the compressed air connection



The supplied flexible and electrically insulating pressure hose between the pipe system and the compressor prevents electricity and vibrations from being transmitted and thus reduces noise. This ensures safe and reliable operation.

1. Connect the pre-assembled connecting sleeve of the pressure hose to the quick release coupling.
2. Measure the required length of the pressure hose and shorten if necessary.
3. Press a fitting hose connector (not included in the scope of delivery) onto the pressure hose (internal diameter 10 mm) and secure it with a hose clip.
4. Connect the connecting sleeve of the pressure hose to the compressed air tube.

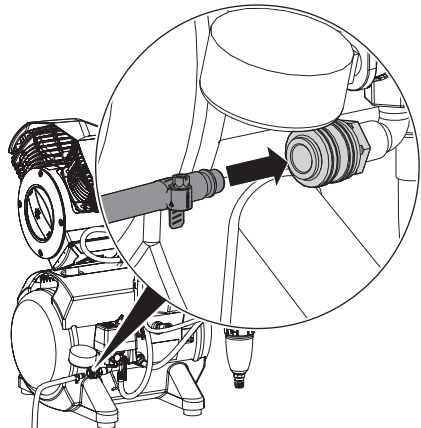

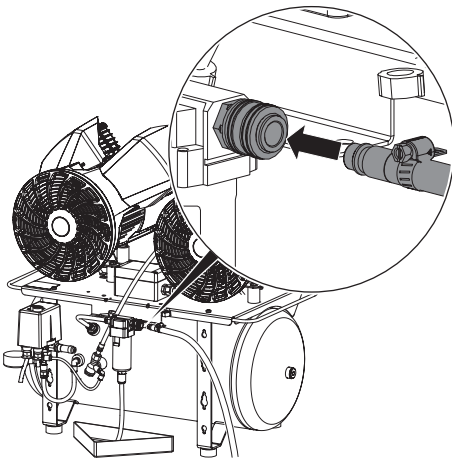


Fig. 1: Tornado 4 Type 42803...

8.3 Place a collector tray underneath

During operation, condensation water on the unit is continuously separated and automatically drained. In order to prevent water damage due to drained condensation, it is collected in the collector tray.



As an option, the condensation can also be drained through a hose into the waste water system. Always comply with applicable national regulations for waste water systems.

1. Place a collector tray under the condensate separator or the membrane drying unit (depending on type).


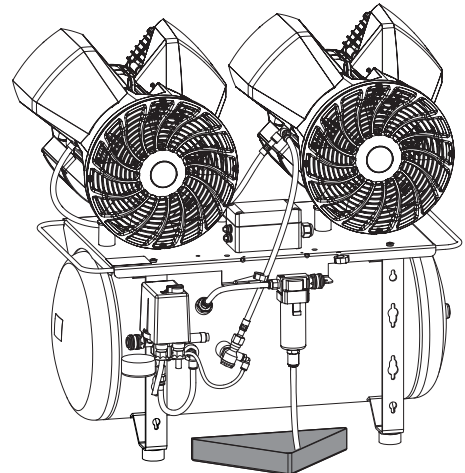


Fig. 2: Tornado 4 Type 42803...

8.4 Electrical connections

Safety when making electrical connections

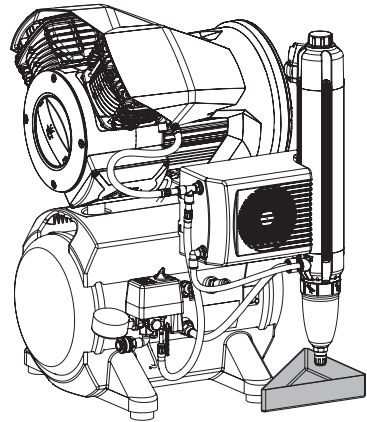

The unit has no main power switch. For this reason it is important that the unit is be set up in such a way that the plug can be easily accessed and unplugged if required.

- The device must only be connected to a correctly installed power outlet.
- Make sure that none of the electrical cables leading to the unit are under any mechanical tension.
- Before taking the unit into operation for the first time, check that the power supply voltage matches the voltage specifications on the type plate.

Establishing the electrical connections




DANGER


Risk of electric shock due to defective mains cable

- › Mains cables must not be allowed to come into contact with any hot surfaces on the unit.

1. Connect the connector of the mains cable to the corresponding mains plug at the pressure switch.

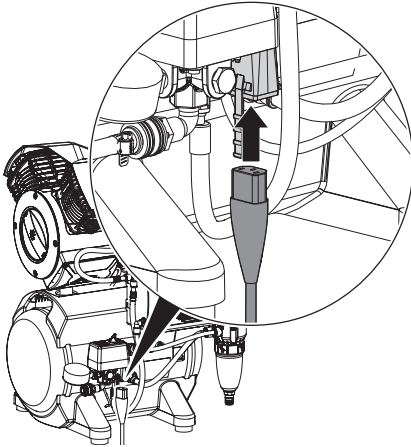

2. Secure the mains connector using the cable tie provided.

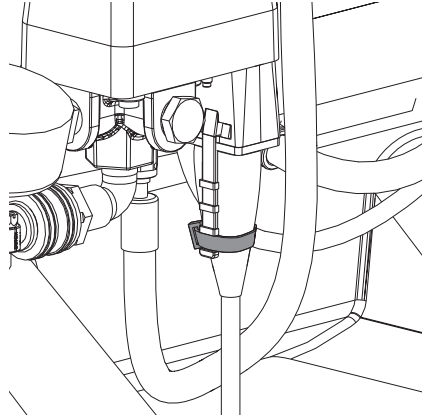

3. Connect the mains plug to an earthed power outlet.

9 Commissioning

1. Before taking the unit into operation, check for any damage. Damaged units must not be used.

 In many countries technical medical products and electrical devices are subject to regular checks at set intervals. The owner must be instructed accordingly.


2. Turn on the unit power switch or the main surgery switch.
3. Carry out an electrical safety check in accordance with applicable local regulations (e.g. the German Ordinance on the Installation, Operation and Use of Medical Devices / Medizinprodukte-Betreiberverordnung) and record the results as appropriate, e.g. in the technical log book.

9.1 Checking the switch-on/cut-off pressure

The switch-on/cut-off pressure is preset at the factory. Check the adjustment during first start-up.

1. Switch on the unit at the pressure switch by turning it to the position "I AUTO".
2. Read off the cut-off pressure from the pressure gauge.
3. Drain the air from the pressure tank (e.g. via the condensate drain valve) until the unit starts and then close it again.
4. Read off the pressure when the unit switches on.


If the readings deviate from the values preset at the factory, adjust the pressure switch to the factory settings.

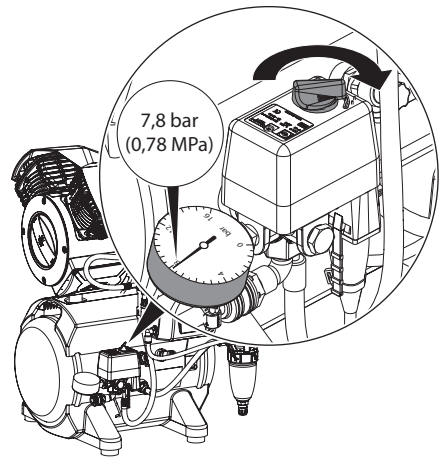

9.2 Draining the condensation water


Temperature changes during transport may cause condensation water to accumulate in the pressure tank. The condensation water can only be drained from the pressurised pressure tank.

1. Switch on the unit at the pressure switch and wait until the cut-off pressure is reached.
2. At maximum tank pressure, slowly open the condensate drain valve.
3. Close the condensate drain valve as soon as all of the accumulated condensation water has been blown out.

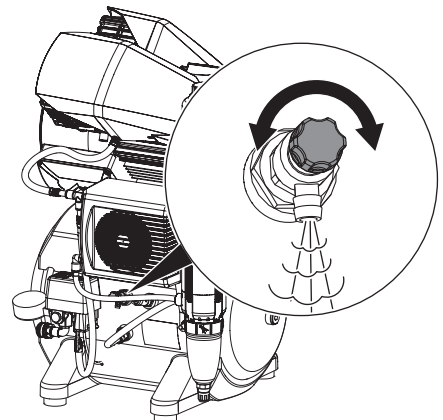

Tornado 4 Type 42803...

1. At maximum tank pressure, unscrew the bottom screw fitting on the condensate separator.
2. Close the screw connection as soon as all of the condensate has been blown out.

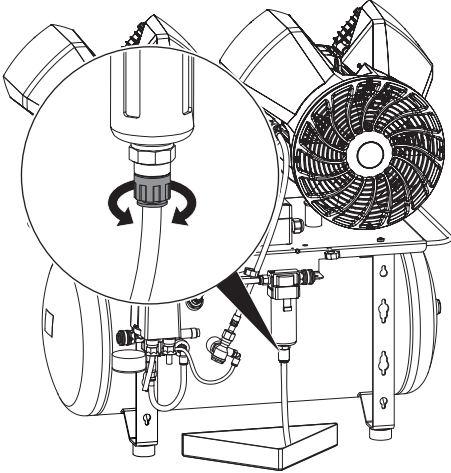

10 Adjustment options

10.1 Adjusting the pressure switch



WARNING

Risk of explosion of the pressure vessel

The pressure vessels used in the compressors are designed to withstand continuous pressure changes of 2 bar and can be used continuously under these pressure changes.

- › For load changes > 2 bar (max. permissible: 3 bar), comply with the maximum load change cycles specified in the operating instructions of the pressure vessel.



DANGER

Exposed live parts

Risk of electric shock due to live parts

- › Disconnect all power from the unit.
- › Use insulated tools.
- › Do not touch live parts.




The cut-off pressure must be at least 0.5 bar (0.05 hPa) below the maximum pressure of 10 bar (1 hPa) of the safety valve. Otherwise the safety valve can open too early, the cut-off pressure will not be obtained, and the compressor generator will run continuously. The maximum permitted pressure is marked by a red line on the attached pressure gauge.

If the read-off values differ from the factory settings or if other settings are required, the cut-off pressure of the compressor can be adjusted at the adjusting screw on the pressure switch. The start-up pressure can then be adjusted using the pressure difference Δp .


1. Take off the pressure switch cover.
2. Adjust the cut-off pressure P at the adjustment screw.
The cut-off pressure increases in the "+" arrow direction and decreases in the "-" arrow direction. The pressure difference Δp is also influenced by this adjustment.

3. Adjust the start-up pressure via the pressure difference Δp at the adjustment screw. The pressure difference increases in the "+" arrow direction and decreases in the "-" arrow direction. The maximum permissible pressure difference must not be set to more than 3 bar.

MDR 1

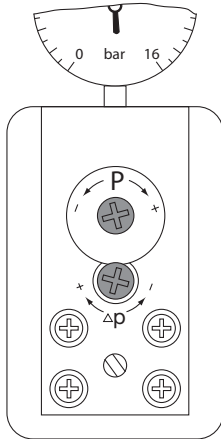


MDR 3

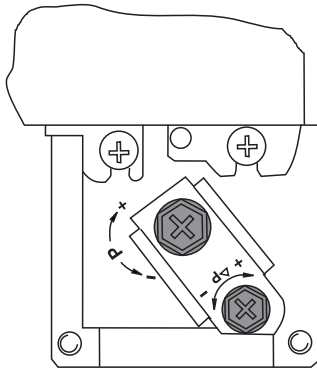


11 Circuit diagrams

11.1 Tornado 1 / Tornado 2

Units without a membrane-drying unit

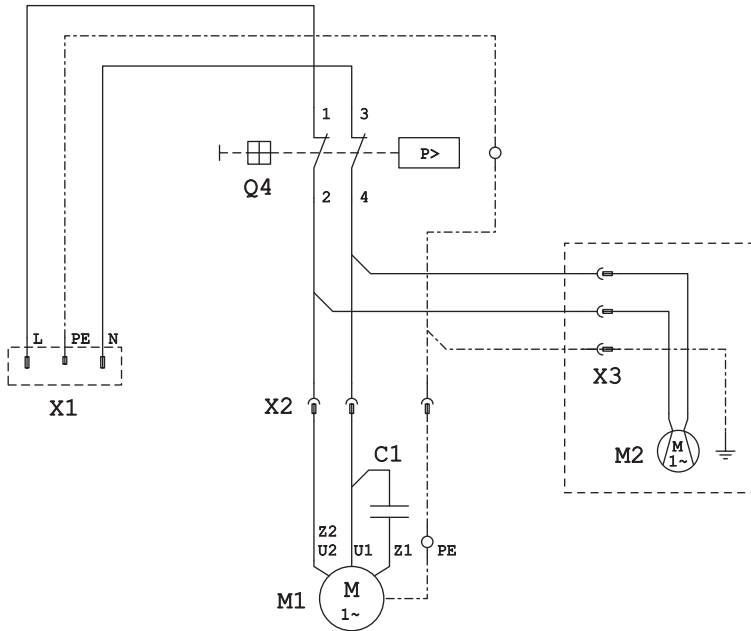


- C1 Condenser
- M1 Compressor motor
- M2 Fan motor, noise reduction hood (if required)
- Q4 Pressure switch
- X1 Mains connection 1/N/PE AC 230 V
- X2 Compressor motor plug connection
- X3 Plug connection for fan motor, noise reduction hood (if required)

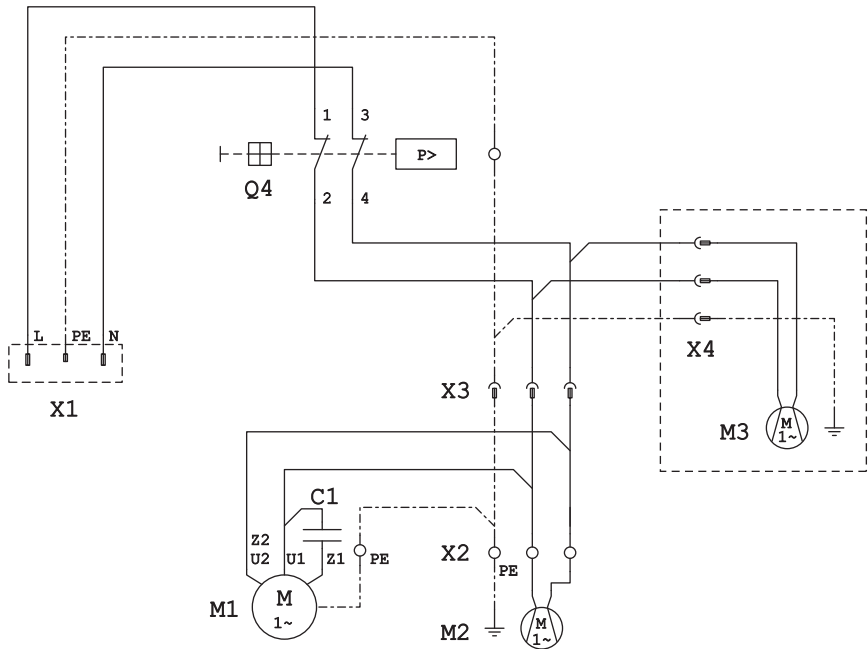Unit with membrane-drying unit


- C1 Condenser
- M1 Compressor motor
- M2 Cooling fan motor, membrane drying unit
- M3 Fan motor, noise reduction hood (if required)
- Q4 Pressure switch
- X1 Mains connection 1/N/PE AC 230 V
- X2 Plug connection for fan motor, cooling, membrane drying unit
- X3 Plug connection for compressor motor and cooling, membrane drying unit
- X4 Plug connection for fan motor, noise reduction hood (if required)

11.2 Tornado 4

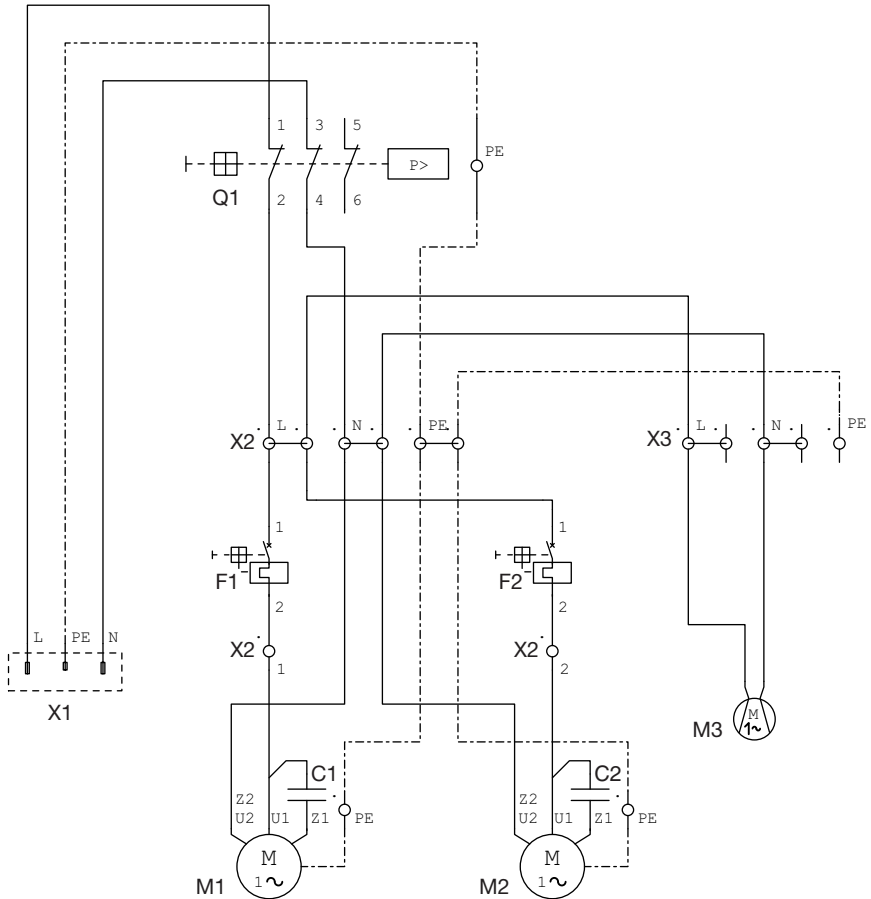

- X1 Mains connection for CEE connector 230 V / 16 A
- X2 Distributor for compressor units
- X3 Distributor for membrane drying unit (type 4282... only)
- Q1 Pressure switch
- F1 Thermal protection switch, nominal current 12 A for M1
- F2 Thermal protection switch, nominal current 12 A for M2
- C1 Motor capacitor
- C2 Motor capacitor
- M1 Compressor unit
- M2 Compressor unit
- M3 Fan motor for membrane drying unit (type 4282... only)

12 Operation




Prior to working on the unit or in case of danger, disconnect it from the mains.

12.1 Switching the unit on/off

1. Switch on the unit at the pressure switch by turning it to the position "I AUTO".

The compressor unit will run automatically and the pressure tank will be filled. When the cut-off pressure has been reached, the compressor unit will switch off automatically.

2. The unit can be switched off when required by turning the pressure switch to the "0 OFF" setting.

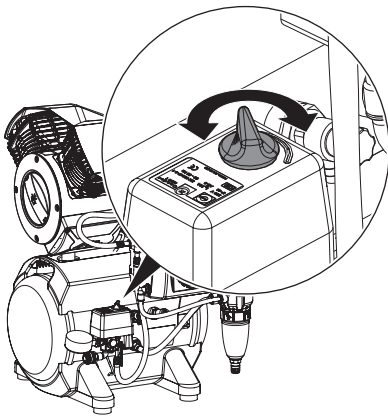

13 Maintenance



Prior to working on the unit or in case of danger, disconnect it from the mains.



WARNING

Risk of infection due to burst filters

Particles enter the compressed air network and can therefore enter the mouth of the patient.

- › Replace filters in accordance with the maintenance schedule.

13.1 Maintenance schedule



NOTICE

Risk of damage to the unit due to blocked filters

Continuous running due to reduced delivery. Damage to the unit due to burst filters.

- › Replace filters in accordance with the maintenance schedule.



Every time you work with the unit, check it visually for damage to ensure safe and reliable operation. Damaged units must not be taken back into use.

Unit without membrane-drying unit

Maintenance interval	Maintenance work
at regular intervals*	<ul style="list-style-type: none"> › Drain off condensed water. › Empty the collector tray under the condensation separator. <p style="margin-left: 40px;">* The interval may vary depending on the ambient conditions and method of working. Empty daily if working in high humidity.</p>
Annually	<ul style="list-style-type: none"> › Replace the air intake filter – every six months if the concentration of dust is high. › Replace the fleece filter in the condensate separator.
Every 5 years	<ul style="list-style-type: none"> › Replace the vibration dampers.
Every 10 years	<ul style="list-style-type: none"> › Change the cup seal.
In accordance with national law	<ul style="list-style-type: none"> › Check the safety valve. › Carry out recurring safety inspections (e.g. pressure tank inspections, electrical safety inspections) in accordance with applicable national laws.


Unit with membrane-drying unit


Maintenance interval	Maintenance work
at regular intervals*	<ul style="list-style-type: none"> › Empty the collector tray under the membrane drying unit. <p style="margin-left: 40px;">* The interval may vary depending on the ambient conditions and method of working. Empty daily if working in high humidity.</p>
Annually	<ul style="list-style-type: none"> › Replace the air intake filter in the compressor unit – do this every six months if there is a high concentration of dust. › Replace fine or virus-bacteria filter. › Replace the coalescence filter. › Check the condensate drain valve on the membrane drying unit.
Every 5 years	<ul style="list-style-type: none"> › Replace the vibration dampers.
Every 10 years	<ul style="list-style-type: none"> › Change the cup seal.
In accordance with national law	<ul style="list-style-type: none"> › Check the safety valve. › Carry out recurring safety inspections (e.g. pressure tank inspections, electrical safety inspections) in accordance with applicable national laws.


13.2 Wear parts and replacement parts

The following working parts need to be changed at regular intervals (refer to the "Maintenance" section):

Air intake filter	5180-982-00
Fine filter	1610-121-00
Virus bacteria filter	1650100172
Coalescence filter	1650200323
Fleece filter	4280-982-00
Vibration damper set Tornado 1	5180100165
Vibration damper set Tornado 2	5280100033
Vibration damper set Tornado 4	4280100045
Cup seal repair set	5180100157


 To configure the required filters or filter sets, you can also use our filter configurator at: www.duerrdental.com/filterkonfigurator

 Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.

 Information about replacement parts can be found on the website portal for specialist dealers under: www.duerrdental.net


13.3 Replacing the air intake filter

1. Switch off the compressor at the pressure switch.
2. Disconnect all power from the device.
3. Release the filter cover by rotating it **anti-clockwise** and then take it off.
4. Remove the air intake filter.
5. Insert a new air intake filter.
6. Place the filter cover in position and lock it by turning it **clockwise**.

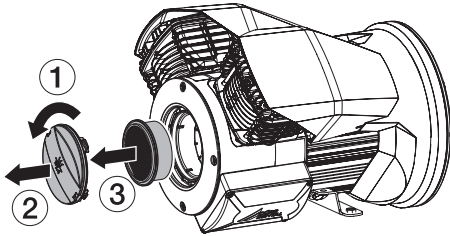


Units with a noise reduction hood


1. Switch off the compressor at the rotary switch of the pressure switch.
2. Disconnect all power from the unit.
3. Take off the noise reducing hood and the foam filter cover.

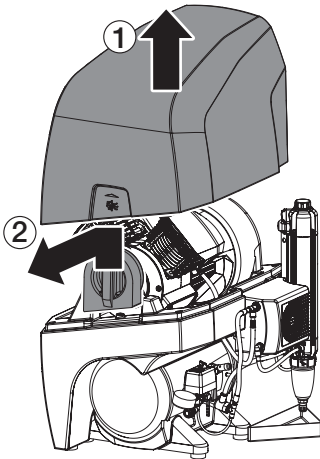

4. Release the filter cover by rotating it **anti-clockwise** and then take it off.

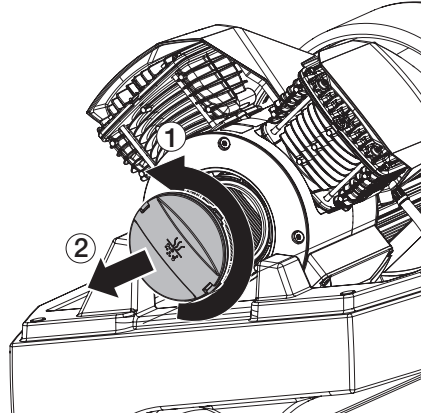


5. Remove the air intake filter.

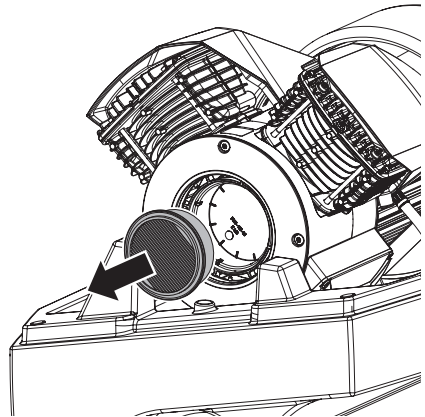


6. Insert a new air intake filter.
7. Place the filter cover in position and lock it by turning it **clockwise**.
8. Fit the the foam filter cover and the noise reducing hood.

13.4 Replace the fleece filter in the condensate separator


1. Switch off the compressor at the pressure switch.
2. Unplug the mains plug.
3. Release the pressure from the pressure tank. To do this, unscrew the bottom screw fitting on the condensate separator.
4. Push the slider of the housing locking mechanism downwards.
5. Open the housing by rotating and pulling downwards.

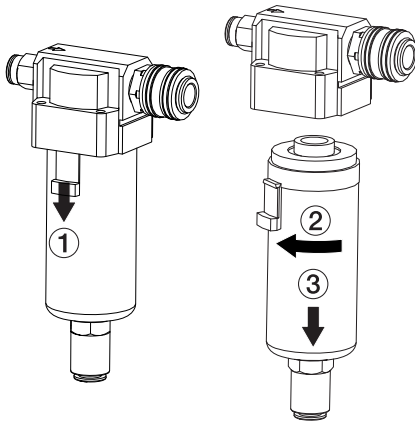

6. Rotate the black filter mount (approx. 45°).
7. Remove the filter mount upwards from the water accumulation chamber.

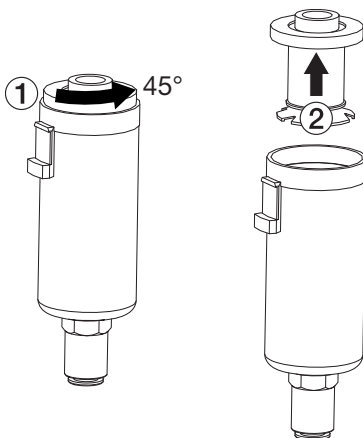

8. Unscrew the rejected part downwards.
9. Remove the filter element.

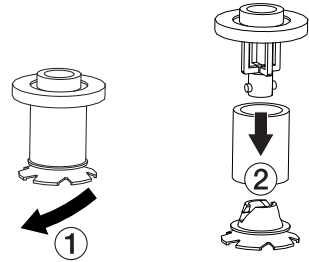

10. Replace the filter element.
11. Re-assemble the condensate separator.




During assembly pay attention to detents and markings.

13.5 Replacing the filter of the membrane drying unit


Fine or virus bacteria filter

1. Unscrew and remove the filter cover.
2. Remove the filter.
3. Insert a new filter.
4. Replace the filter cover and close.

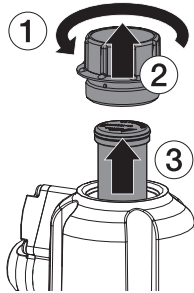

Coalescence filter

1. Unscrew and remove the filter housing.
2. Remove the filter.
3. Replace O-ring.
4. Insert a new filter.
5. Replace the filter housing and close.

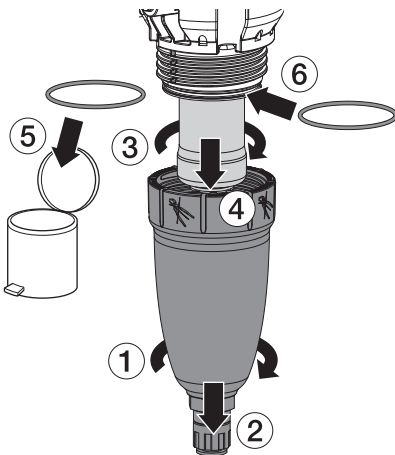

13.6 Checking the safety valve

The functioning of the safety valve must be checked at regular intervals in accordance with national regulations.



WARNING


Risk of damage to the safety valve

Risk of explosion of the pressure tank and pressure hoses due to a defective safety valve


- Do not use the safety valve to vent the pressure tank.

1. Switch on the unit at the pressure switch and fill the pressure tank to the cut-off pressure.
2. To open, rotate the screw of the safety valve anti-clockwise until the valve begins to blow off. Only allow the safety valve to blow for a short period.
3. Then turn the screw clockwise as far as it will go to close the valve. The valve must now be closed again.

Tornado 1 / Tornado 2

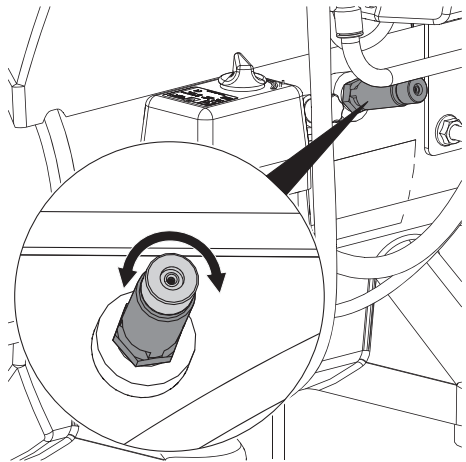

Tornado 4

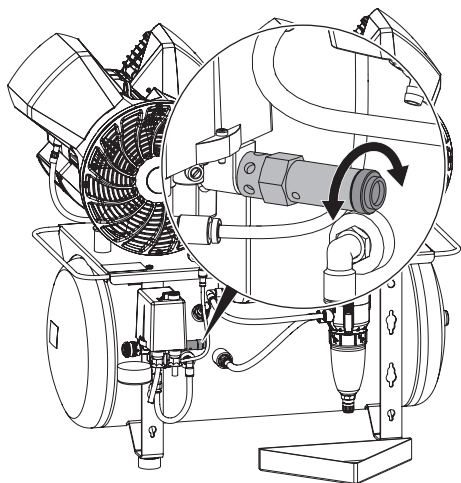

14 Taking out of use

14.1 Taking the unit out of use

If the unit is not to be used for a prolonged period of time, we recommend that it is properly shut down and taken out of use.

To do this, the accumulated condensation water from the unit must be drained.


1. Switch on the unit and wait until the cut-off pressure is reached.

Condensate separator

1. Open the condensate drain valve on the condensate separator.


Once the start-up pressure has been reached the compressor will switch on.

2. With the compressor switched on and the condensate drain valve open, wait until no more condensation water emerges.
3. Switch off the device.
4. Close the condensate drain valve when no more air emerges.
5. Disconnect the mains plug.
6. Disconnect the compressed air connection on the quick release coupling.

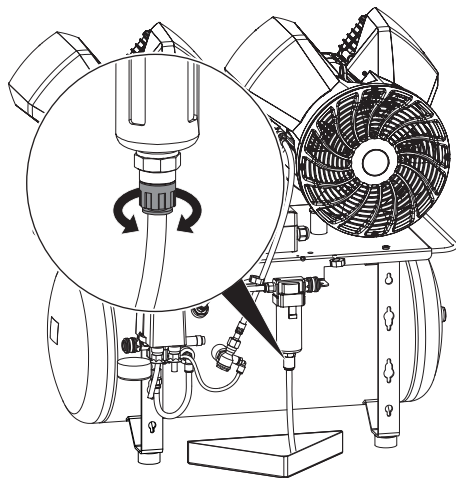


Membrane drying unit

1. Open the condensate drain valve on the membrane drying unit with the compressor unit running. When no more condensation water emerges, close the condensate drain valve.
2. Switch off the device.

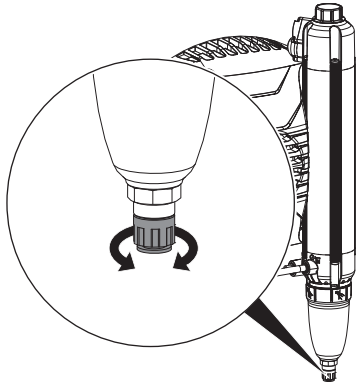

Pressure tank

1. Open the condensate drain valve.
Once the start-up pressure has been reached the compressor will switch on.
2. With the compressor switched on and the condensate drain valve open, wait until no more condensation water emerges.
3. Switch off the device.
4. Close the condensate drain valve when no more air escapes.
5. Disconnect all power from the unit.
6. Disconnect the compressed air connection on the quick release coupling.

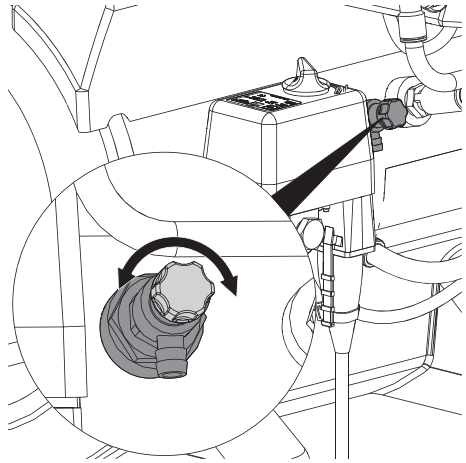

14.2 Storage of the unit



WARNING

Risk of explosion of the pressure tank and pressure hoses

› The pressure tank and the pressure hoses must be vented before they are stored or transported.

- Protect the unit against moisture, dirt and extreme temperatures during storage (refer to the section on "Ambient conditions").
- Only store the unit when it has been completely emptied.

Troubleshooting

15 Tips for operators and service technicians



Any repairs exceeding routine maintenance may only be carried out by qualified personnel or our service.



Prior to working on the unit or in case of danger, disconnect it from the mains.

Error	Possible cause	Remedy
Compressor will not start	No mains voltage	<ul style="list-style-type: none"> › Check the mains fuse; if necessary, switch the circuit breaker back on. If the fuse is defective, replace it.
	Undervoltage or overvoltage	<ul style="list-style-type: none"> › Measure the supply voltage; call an electrician if necessary.
	Pressure switch not switched on	<ul style="list-style-type: none"> › Switch on the pressure switch. › Inform a service technician.
	Motor winding overheating protector tripped (overheating)	<ul style="list-style-type: none"> › Allow the unit to cool down. › Inform a service technician.
	Thermal protection switch tripped	<ul style="list-style-type: none"> › Inform a service technician. › Reset the protection switch.
Humming noise from motor	Motor capacitor is defective	<ul style="list-style-type: none"> › Replace the capacitor.
Compressor does not switch off	Wrong size of compressor, air intake too high	<ul style="list-style-type: none"> › Calculate the air requirement (this can be up to 50 l/min per treatment unit), if necessary install a larger compressor.
	Leak in the compressed air system	<ul style="list-style-type: none"> › Locate and seal the leak. › Inform a service technician.
	Defective membrane drying unit	<ul style="list-style-type: none"> › Check whether there is an increased flow of air at the filter housing of the membrane drying unit (bottom), if necessary replace the membrane drying unit.
Compressor switches on from time to time even though no air is being taken for a consumer unit	Leak in the compressed air system	<ul style="list-style-type: none"> › Locate and seal the leak. › Inform a service technician.
Knocking or loud noises on the compressor	Compressor unit defective	<ul style="list-style-type: none"> › Disconnect all power from the device and inform a service technician.

Error	Possible cause	Remedy
Reduced delivery. Compressor needs longer to charge the pressure tank, see charging times in "4 Technical data"	Air intake filter dirty	› Replace the air intake filter at least 1x per year. The air intake filter must never be cleaned.
	Defective membrane drying unit	› Replace the membrane drying unit. › Inform a Service Technician.
	Cup seal worn at the piston or defective	› Replace the cup seal or the entire piston.
Water dripping from air consumers	Maintenance work not carried out regularly (without membrane drying unit)	› Regularly drain the condensation water from the pressure tank, see "9.2 Draining the condensation water"
	Defective membrane drying unit	› Inform a service technician.
Working cycles of the compressor are very short, even if only small amounts of air are removed	Condensed water in the tank	› Draining off condensed water › The dry air units of compressors fitted with dry air units need to be checked and replaced, if applicable.

 Appendix

16 Handover record

This document confirms that a qualified handover of the medical device has taken place and that appropriate instructions have been provided for it. This must be carried out by a qualified adviser for the medical device, who will instruct you in the proper handling and operation of the medical device.

Product name	Order number (REF)	Serial number (SN)

- Visual inspection of the packaging for any damage
- Unpacking the medical device and checking for damage
- Confirmation of the completeness of the delivery
- Instruction in the proper handling and operation of the medical device based on the operating instructions

Notes:

Name of person receiving instruction:**Signature:**

Name and address of the qualified adviser for the medical device:

Date of handover:**Signature of the qualified adviser for the medical device:**

--	--

17 Country representatives

Country	Address
GB	 UK Responsible Person: Duerr Dental (Products) UK Ltd. 14 Linnell Way Telford Way Industrial Estate Kettering, Northants NN 16 8PS
UA	 UA.TR.099 Уповноважений представник в Україні: Приватне підприємство “Галіт” вул. 15 квітня, 6Є, с. Байківці, Тернопільський р-н, 47711, Україна тел.: 0800 502 998; +38 050 338 10 64 www.galit.te.ua; e-mail: office@galit.te.ua Виробник: Дюерр Дентал ЕсЕ Хьолфгхаймер Штрассе 17, Д-74321 Бітгхайм-Біссінген, Німеччина email: info@duerrdental.com



Hersteller / Manufacturer:

DÜRR DENTAL SE
Höfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
Fon: +49 7142 705-0
www.duerrdental.com
info@duerrdental.com

